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Left Ear

Bacteria, Cocci

3-4+ Numerous coccoid-shaped bacteria seen.

Bacteria, Rods

2+ Moderate rod-shaped bacteria seen.

Yeast

2+ Moderate yeast seen.

WBC

Absent

Mites

Absent

Diagnostic Considerations

Results: Both bacteria and yeast seen. Consideration: The co-presence of yeast and bacteria should be interpreted in light of clinical signs and degree of burden. Low numbers of bacteria and yeast may represent commensal flora in unaffected ears, early dysbiosis in mildly affected ears, or true infection in clinically affected ears. As the bacteria and yeast burden increases, and if white blood cells are also present, true infection becomes more likely. Next Step(s): Address any primary and predisposing factors—such as atopic dermatitis, endocrinopathies, excessive hair in the ear canals, stenotic ear conformation, increased cerumen production, otic masses, or frequent ear cleaning. In cases of persistent or recurrent infections, especially those with pus or discharge, evaluate the patient for the presence of biofilms, which can make organisms resistant to antimicrobials and require thorough ear cleaning as part of treatment. Use clinical signs, history, and diagnostics to evaluate for deeper involvement of the middle or inner ear. Administer appropriate antimicrobial and anti-inflammatory therapies if warranted based on clinical assessment.

Images

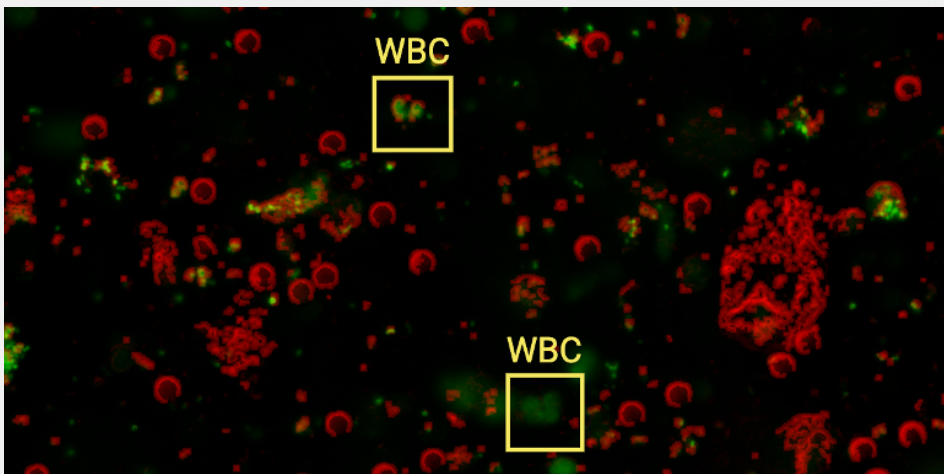
Visit type: First time. Case duration: Chronic. Clinical signs: Patient presented signs of otitis including:

• Left ear: itchy, pain, erythema (redness), discharge (waxy, watery).

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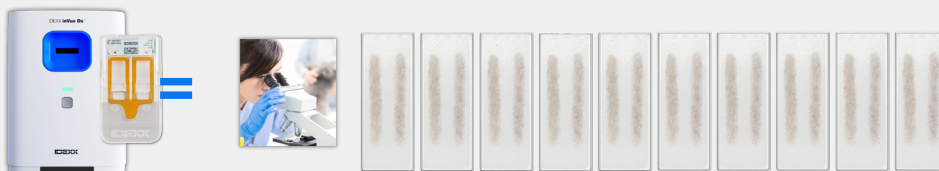
The screenshot shows the IDEXX VetLab Station interface. On the left, a patient profile for Bindi Brooks (959935) is displayed with fields for Test Order (Pathology), *Req ID (54674636), *Confirm Life Stage (Adult Canine), Doctor (Dr. Polly Suarez), *Weight (lbs) (80), and Reason for Testing (Wellness Testing, Sick Testing, Monitoring, Pre-Anesthesia). The main area shows a list of test orders: Catalyst One (Chemistry, Busy), SNAP Pro (Immunoassay, Ready), SNAP (Immunoassay, Manual), IDEXX ProCyte Dx (Hematology, Busy), and IDEXX inVue Dx (Pathology, Ready). The IDEXX inVue Dx order is expanded, showing 'Ear Cytology' and options to 'Add case notes to report (optional)' with buttons for 'First time', 'Recheck exam', 'Acute', 'Chronic', and 'Add clinical signs'.

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The screenshot shows the IDEXX VetLab Station interface with the 'Optional Clinical signs' dialog box open. The dialog box has a title 'Optional Clinical signs' and a subtitle 'Select any that apply (optional):'. It lists several clinical signs: Itchy, Erythema (redness), Ulcers, Swelling, Pain, Odor, and Discharge. Each sign has three buttons: 'Left', 'Both', and 'Right'. The 'Discharge' button is highlighted with a blue arrow. At the bottom of the dialog box are buttons for 'Cancel', 'Clear selections', and 'Save'.

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
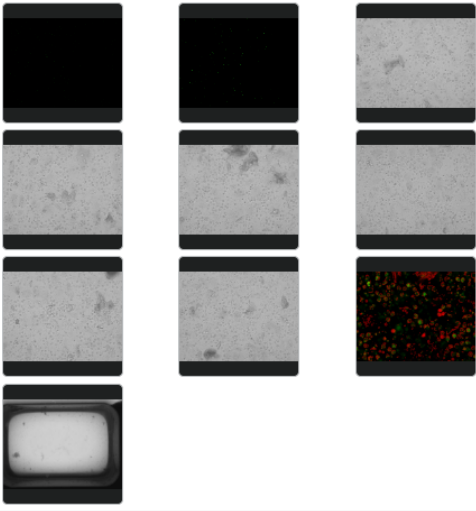
Pathology		3/26/26 3:16 PM
Site:	Left Ear	
Bacteria, Cocci	3-4+ Numerous coccoid-shaped bacteria seen.	
Bacteria, Rods	2+ Moderate rod-shaped bacteria seen.	
Yeast	1+ Few yeast seen.	
WBC	Present	
Mites	Absent	
Diagnostic Considerations		
Results: Both bacteria and yeast seen. Consideration: The co-presence of yeast and bacteria should be interpreted in light of clinical signs and degree of burden. Low numbers of bacteria and yeast may represent commensal flora in unaffected ears, early dysbiosis in mildly affected ears, or true infection in clinically affected ears. As the bacteria and yeast burden increases, and if white blood cells are also present, true infection becomes more likely. Next step(s): Address any primary and predisposing factors—such as atopic dermatitis, endocrinopathies, excessive hair in the ear canals, stenotic ear conformation, increased cerumen production, otic masses, or frequent ear cleaning. In cases of persistent or recurrent infections, especially those with pus or discharge, evaluate the patient for the presence of biofilms, which can make organisms resistant to antimicrobials and require thorough ear cleaning as part of treatment. Use clinical signs, history, and diagnostics to evaluate for deeper involvement of the middle or inner ear. Administer appropriate antimicrobial and anti-inflammatory therapies if warranted based on clinical assessment. Results: White blood cells present. Consideration: Consider underlying causes of otitis externa—such as atopic dermatitis (food or environmentally triggered), tumor, otitis media, foreign body presence, or contact otitis as from otic cleaners/medications or aggressive mechanical cleaning. Next Step(s): In ears undergoing treatment, persistent inflammation indicates the need to investigate for an underlying cause. Typically, these patients require more intensive/longer duration of treatment and more intensive diagnostics (otic irrigation, advanced imaging, and sometimes otic culture). Address any primary, predisposing, and perpetuating factors as part of the therapeutic regimen.		
Images		
		
Visit type: First time. Case duration: Acute. Clinical signs: Patient presented signs of otitis including: • Left ear: itchy, pain, odor, erythema (redness).		

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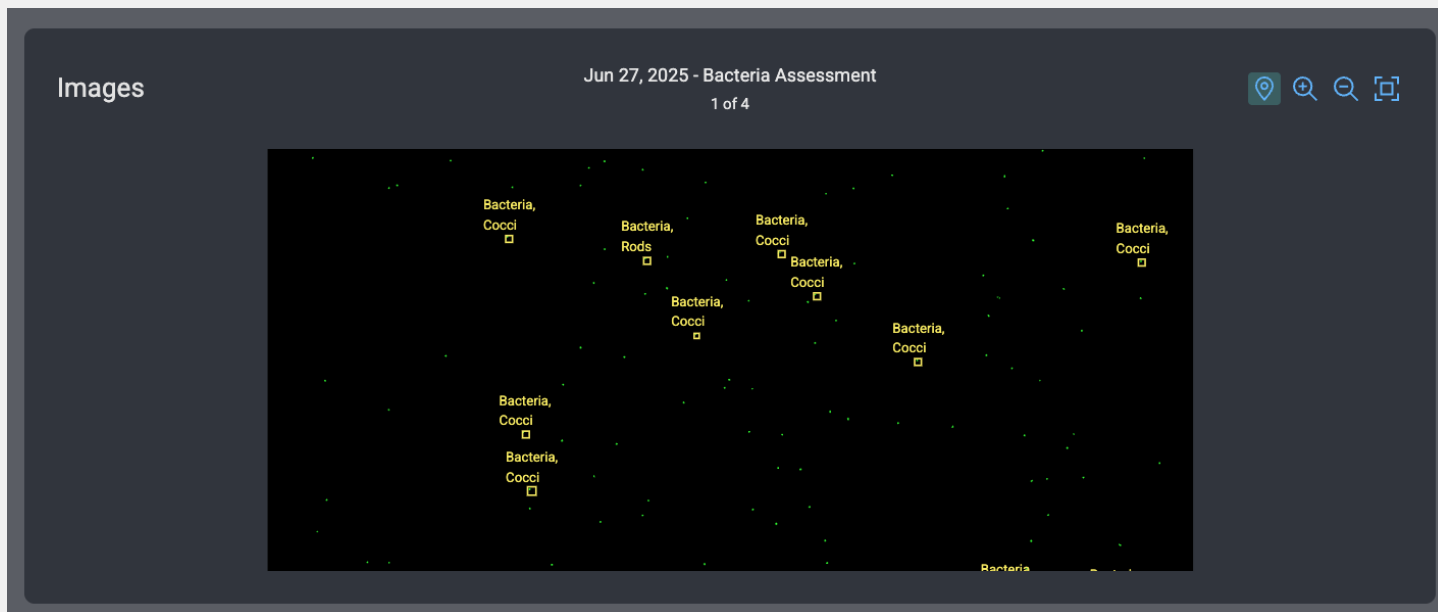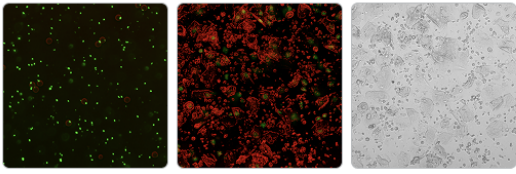
the presence of biofilms, which can make bacteria resistant to antibiotics and require thorough ear cleaning as part of treatment. Use clinical signs, history, and diagnostics to evaluate for deeper involvement of the middle or inner ear. Administer appropriate antimicrobial and anti-inflammatory therapies based on clinical assessment.

User has indicated that there is purulent discharge in this ear. Consider suppurative otitis: clinically characterized by erythema, ulceration, and a purulent discharge often with a biofilm. These cases usually involve a *Pseudomonas* spp. infection but can rarely be associated with *Staphylococcus* or *Malassezia*. Address potential primary, predisposing, and perpetuating factors such as atopic dermatitis (food or environmentally triggered), tumor, otitis media, foreign body presence, infection and its potential extension to the middle ear, and address biofilm as part of your elected treatment as these protect bacterial colonies from antimicrobial therapy.

Images


Bacteria Assessment Yeast and WBC Assessment (Composite) Yeast and WBC Assessment (Brightfield)

Visit type: First time. Case duration: Chronic. Clinical signs: Patient presented signs of otitis including:
Left ear: pain, itchy, discharge (purulent (pus), bloody)

 **Pathology** 5/14/25 8:02 AM 

Source	Right ear	
Bacteria, Rods	0	None
Bacteria, Cocci	0	None to trace seen
Yeast	0	None to trace seen
WBC	Absent	
Mites	Present	
Diagnostic Considerations	Results: Otodectes otitis. Consideration: Any co-presence of bacteria, yeast, and/or white blood cells is likely secondary to ear mite infestation. Next Step(s): Use an effective acaricide and manage any secondary infections or inflammation.	

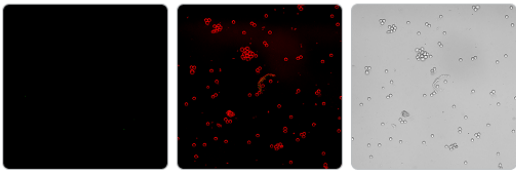Images





Bacteria Assessment Yeast and WBC Assessment (Composite) Yeast and WBC Assessment (Brightfield)

Visit type: First time. Case duration: Chronic. Clinical signs: Patient presented signs of otitis including:
Left ear: pain, itchy, discharge (purulent (pus), bloody)

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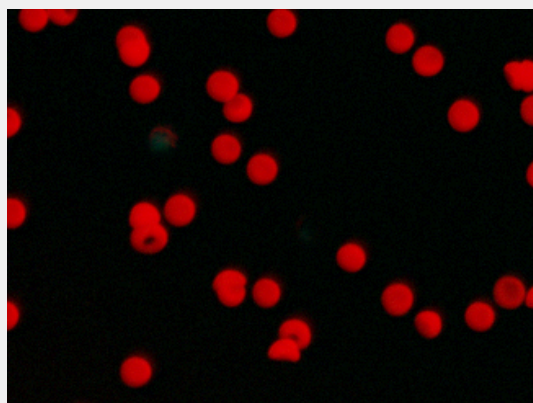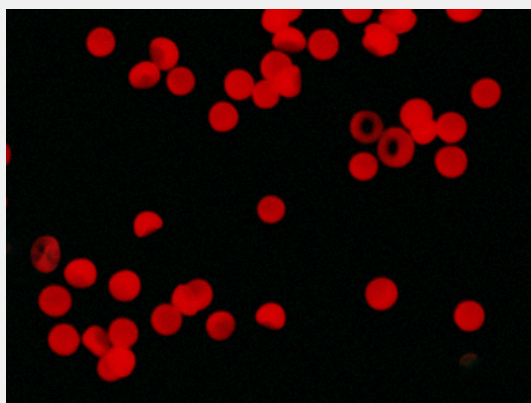
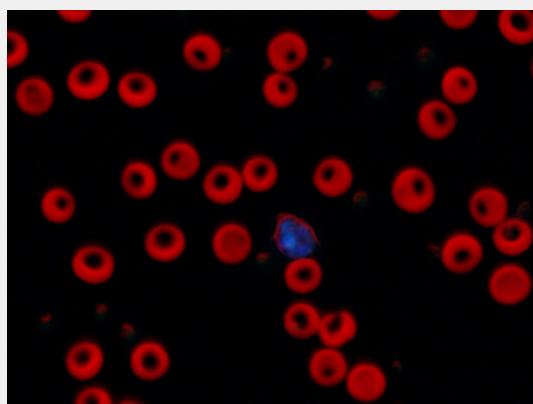
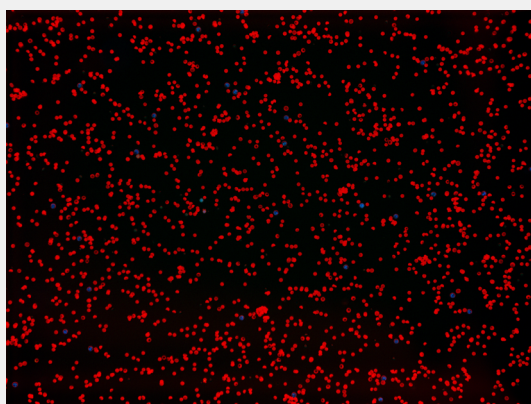
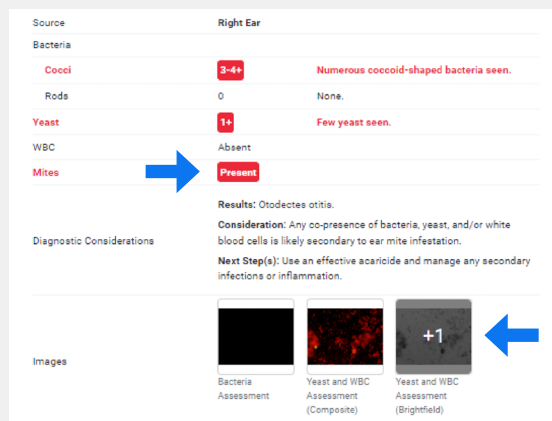
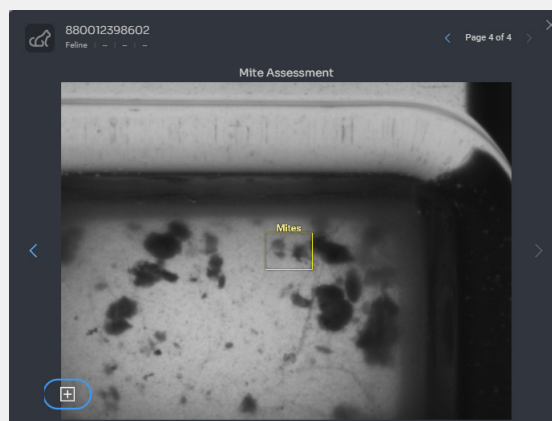
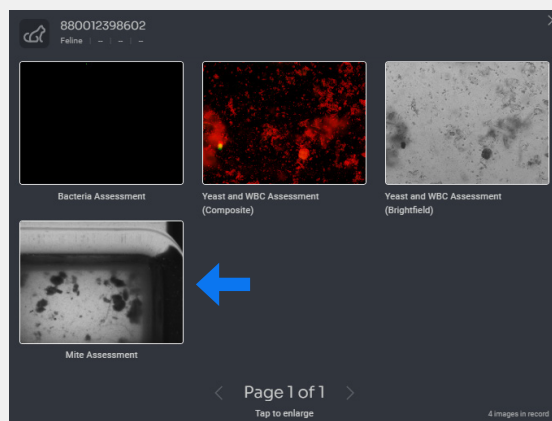
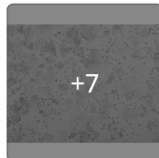
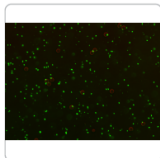
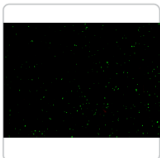
Results Details ▾   [Manage Results](#)

WBC	Absent
Mites	Absent
Diagnostic Considerations	Results: Both rods and cocci observed Consideration: The co-presence of rods and cocci support bacterial otitis. Next Step(s): In cases of persistent or recurrent infections, especially those with pus or discharge, evaluate the patient for the presence of biofilms, which can make bacteria resistant to antibiotics and require thorough ear cleaning as part of treatment. Use clinical signs, history, and diagnostics to evaluate for deeper involvement of the middle or inner ear. Administer appropriate antimicrobial and anti-inflammatory therapies based on clinical assessment.
Images	 Bacteria Assessment Bacteria Assessment Yeast and WBC Assessment (Brightfield)
Source	Right Ear



 **Buxton Brooks**
Canine | Poodle | Female | 4 y ×

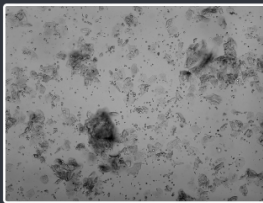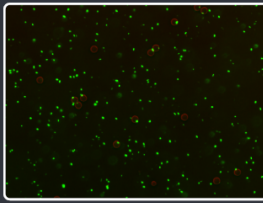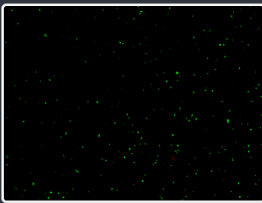Bacteria Assessment

Bacteria Assessment

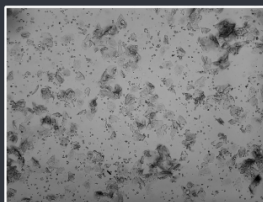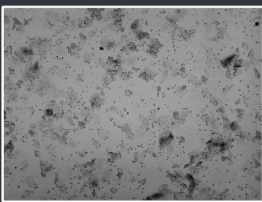Yeast and WBC Assessment (Brightfield)

Yeast and WBC Assessment (Brightfield)

Yeast and WBC Assessment (Brightfield)

Yeast and WBC Assessment (Brightfield)

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© 2026 IDEXX Laboratories, Inc. Todos los derechos reservados. • 09-2692859-00
1. Datos en archivo de IDEXX Laboratories Inc. Westbrook, Maine (Estados Unidos).