## IDEXX VetStat\*

Peristaltic Pump and Gas I/O Port Kit

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06-06614-00

## IDEXX VetStat\* Analyzer Peristaltic Pump and Gas I/O Port Kit

The peristaltic pump cartridge and gas I/O port seal must be replaced annually to ensure that your VetStat analyzer operates at peak performance.


**Contents:** One peristaltic pump cartridge, two pump seals and one gas I/O port seal

### Replace the Peristaltic Pump Cartridge


1. Turn off the VetStat analyzer and open the printer cover door. The peristaltic pump is located to the right of the printer.
2. Remove the pump by firmly grasping the housing and pulling upward (Figure 1).
3. Using tweezers or hemostats, carefully pull out the two pump seals and discard them.
4. Insert the two new seals, making sure that the small diameter holes (flat top) are face down. Ensure that the seals are fully seated.

**NOTE:** Seals are fully seated when the front seal is flush with the surface and the rear seal is approximately 1/8" (3 mm) below the surface.

5. With the nipples facing down, hold the new peristaltic pump cartridge next to the pump motor. Turn the motor shaft by hand so that it aligns with the key-way of the pump spool. Then, align the pump cartridge with the motor shaft and push the cartridge down firmly until it is fully seated on the housing (Figure 2).
6. Press the pump cartridge roller down until it is firmly seated on the shaft of the pump motor (Figure 2).

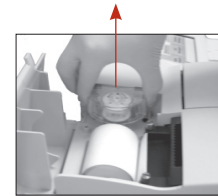**Figure 1:** Remove the peristaltic pump cartridge

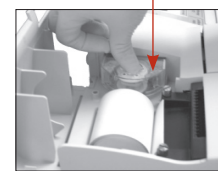**Figure 2:** Install the new peristaltic pump cartridge and then push down the pump roller

### Replace the Gas I/O Port Seal

1. Open the sample measurement chamber (SMC) cover. Remove the black gas I/O port seal (Figure 3) by grasping it with a hemostat or tweezers and firmly pulling upward. Discard the gas I/O port seal.

**WARNING:** Do not push the hemostat or tweezers down into the port opening as this may cause damage to the analyzer.

2. Install the new gas I/O port seal with the rounded surface pointing up and press it into the recess. When fully seated, the I/O port seal is approximately 1/8" (3 mm) above the surrounding surface.

Cassette seat Gas I/O port


**Figure 3:** Inside the sample measurement chamber

### Test the Pump Motor

This test ensures the correct operation of the peristaltic pump.

1. From the Home screen, tap **System**, tap **Diagnostics** and then tap the **Tests** tab.
2. Tap **Pump** (Figure 4) to test the pump motor. The pump should run smoothly without excessive noise. This diagnostic test automatically steps through all the speeds used during normal operation (50–800 rpm) and then returns to the Diagnostics > Tests tab.

When the diagnostic test is complete, run one sample in control mode, ensuring that the control measurement passes without errors. (See the VetStat Analyzer Operator's Guide for more information.)

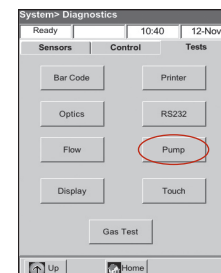**Figure 4:** System > Diagnostics > Tests > Pump

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\*VetStat es una marca de IDEXX Laboratories, Inc. en los Estados Unidos de América y/o en otros países. © 2005 IDEXX Laboratories, Inc. Todos los derechos reservados.